# maxTV®

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**User Manual**

**Mode d'emploi**

**Istruzioni d'uso**

**Manual de instrucciones**

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**Bruksanvisning**

**Käyttöopas**

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## English

Your new acquisition is a quality product from the house of Eschenbach. This top-class 'Made in Germany' product was manufactured using the most up-to-date production methods available. We congratulate you on your choice.

### Intended use

The maxTV glasses were designed especially for watching television. The focus was on user-friendliness, strong magnification of the television image in good quality, and a modern design.

### Safety information

- **Risk of fire!**  
Lenses used in optical equipment may cause considerable damage through their 'burning glass effect' if used or stored incorrectly! Ensure that optical lenses are never left uncovered in the sunlight!
- **Danger of blinding or injury!**  
Never look at the sun or other very bright sources of light through optical equipment!
- **Make sure that other people and especially children are aware of this!**
- **The maxTV is only suitable for watching television and must not be used for driving!**
- **Protect your maxTV from impact and excessive heat! Never place the maxTV on heaters or in sunlight!**
- **The product is a Class I medical device (non-sterile, without measuring function) and complies with Regulation (EU) 2017/745. For use of the product outside the EU member states, the respective country-specific directives must be observed.**

### Earpiece adjustment

The plastic earpieces ② can be bent in the desired position without warming. An internal metal wire ensures that the earpieces remain in this position.

### Adjustment

The optical characteristics of the maxTV have been calculated for a distance of 3 m. The maxTV offers a diopter adjustment (option for correcting defective vision) of  $\pm 3$  dpt. It can be adjusted for each eye individually. You can also use the diopter adjustment to adapt the glasses to other distances besides 3 m.

Position yourself at the desired distance from the television and put on your maxTV.

Keep your left eye closed and turn the knurled ring ① until the television image seen by your right eye is in sharp focus.

Do the same with the other eye.

### Notes

Under the following circumstances, it might not be possible to see the television image perfectly:

- If the maxTV is bent open too far, for example due to an unusually large head, doubled images might occur, because the optical axes of the two lens systems are not in parallel any more.
- A very small or very large pupil distance (eye distance) can reduce the performance quality, because the image is not viewed through the optical zones of the maxTV any more.
- If you normally have glasses with diopters beyond  $\pm 3$ , the diopter adjustment feature of the maxTV cannot entirely compensate the defective vision. In this case, change your distance to the TV until the image is in focus.
- If you usually wear glasses with cylindrical lenses (e. g. for astigmatism),

it might not be possible to achieve perfect focus. This depends on the strength of the cylinder.

## Care instructions

maxTV is a high-quality, technologically advanced, complex optical aid and needs to be handled and cared for accordingly. For protection against dust and external influences, we recommend that you store it in the case that is included in the scope of supply.

- **Do not use alcohol or organic solvents to clean the maxTV!**
- **Do not clean MAX TV in an ultrasonic bath!**

Clean the maxTV with a soft cotton or line cloth (e. g. a glasses cloth). Moisten the cleaning cloth to remove more persistent stains (e.g. finger marks).

## Technical data

|                                              |                                        |
|----------------------------------------------|----------------------------------------|
| Article number                               | 162411                                 |
| Diopter adjustment for each eye individually | $\pm 3$ dpt                            |
| Magnification                                | 2.1 $\times$                           |
| Field of vision                              | $18^\circ = 1 \text{ m} / 3 \text{ m}$ |
| Pupil distance                               | 60 mm – 68 mm                          |
| Weight                                       | 49 g                                   |

## Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

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Eschenbach Optik GmbH